# Tiger Gel Clear

Lichthärtender Reparatur- und Modellierkunststoff  
Light cure repair and modeling resin

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**Dear doctor,** we are honoured that you have chosen this innovative new product from Tiger Dental GmbH.

To ensure the safe and effective use of this product, you must carefully read and adhere to this user manual.

The user manual covers all necessary safety instructions but cannot describe all possible circumstances. Therefore, you will also find video clips on our website [www.tigerdental.com](http://www.tigerdental.com) to enable an audio-visual description of the purpose of use, mentioned in this manual. Additionally, our customer service is available for any additional questions.

The continual development of our products indicates our company's commitment to innovation. Accordingly, after several clinical uses, we recommend you carefully review the accompanying user manual (this can be found online under the "Downloads" link at [tigerdental.com](http://tigerdental.com)).

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## 3. Product description

1. Tiger Gel is a light cure resin gel for use in dental practice and dental laboratories. It is used in orthodontics to coat all kind of metal parts in the mouth to make them smoother and enhance the experience and comfort for the patient.

2. It is easy and precise to apply and therefore sparingly and not time consuming. And the material is immediately positionally stable.

3. Tiger Gel cures in the wavelength range between 400 and 500 nm and can be cured with commercially available light curing units.

## 4. Indications

- Coating of metal parts (e.g. wires, bands, brackets) in the mouth
- Creation of bite blocks
- Reattaching teeth to dentures

## 5. Application

Before applying the Tiger Gel to the corresponding metal parts or plastic surfaces, these must be cleaned thoroughly.

The Tiger Gel is not self-adhesive. This enables the user to easily remove it. If additional support is required, the surfaces to be treated must also be roughened. Depending on the power of the light curing lamp, the Tiger Gel takes 20-180 seconds to harden.

Remove the syringe cap and lock one of the provided NeedleTips. NeedleTips are for single use only! After use replace the syringe cap.

Tiger Gel is applied directly from the syringe. To ensure complete curing seek for a constant layering (ca. 1-2 mm) while applying the material. Cure each layer separately.

Curing time depends on the used curing unit:

Heraeus Kulzer Unix SX: 180 s

Heraeus Kulzer Translux: 40 s

## 6. Storage instructions

Do not store above 25°C (77°F). Protect from direct sunlight. Do not use after expiry date.

Close syringe immediately after use to avoid exposure to light.

## 7. Safety Instructions

- Avoid contact of uncured material with eyes and skin.
- If the material comes into contact with skin, immediately wash with water and soap. If the material comes into contact with eyes, immediately rinse with copious amounts of water and seek medical advice if required.
- On sensitive persons, sensitization to the material cannot be excluded. The material should not be used any more, if allergic reactions are observed.
- Do not use for patients with allergic reactions against acrylate.
- Do not breathe polishing dust; use suitable mouth protective device or aspiration!

## 8. References of Quality

Tiger Dental assures the user a flawless quality of its products. The content of this manual is based on experience. The user is responsible for the processing of the products themselves. Tiger Dental is not liable for erroneous results or processing by the user that occur in the absence of influence of Tiger Dental. Although this product is a mature product, improvements and changes can be made, which may not be documented in the present manual. Similarly, we are not liable for printing errors.

## 9. Contents

1 ml syringe Light Curing Resin

5 needle tips gauge 20

## 10. Description of the used symbols of ÖNORM EN ISO 15223-1

 European Conformity

 Manufacturer

 Date of manufacture

 Lot code

 Item number

 Attention! Follow the instructions for use

 Expiry Date

 25 °C  
Storage Conditions - Upper Temperature limit

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Manufacturer: S&C Polymer, 25335 Elmshorn, Germany

Tiger Dental GmbH, Belruptstr. 59,  
6900 Bregenz, Austria, EU  
[www.tigerdental.com](http://www.tigerdental.com)